

Same quaternary ammonium based formula as market leading germicidal wipe brands

Same wipe size as market leading germicidal wipe brands

Registered with the U.S. Food and Drug Administration (FDA), 2020, as a disinfectant for medical devices and surfaces

- + Two-minute dry time at room temperature
- + Medical grade surface wipes for hospital and healthcare settings, and critical care areas
- + Disinfect medical diagnostic equipment and hard, non-porous surfaces
- + 160 pre-saturated wipes per container
- + For use on non-surgical medical devices only

To disinfect and deodorize:

To disinfect non-food contact surfaces only: Unfold a clean wipe and thoroughly wet surface. Allow treated surface to remain wet for two (2) minutes. Let air dry. For heavily soiled areas, use a wipe to pre-clean prior to disinfecting.

Wynn Med Inc.
Sales@WynnMed.com
wynnmed.com

**TWO
MINUTE
DRY TIME!**



Medical Grade Surface Sanitizing Wipes

We know it's more imperative than ever to disinfect medical diagnostic equipment and high-touch, non-porous surfaces as quickly and efficiently as possible. Given the incredibly high demand for sanitizing wipes due to the spread of COVID-19, we are proud to offer new germicidal wipes *with a dry time of just two minutes*.

Made for healthcare and hospital setting use, these fast-acting wipes allow facility staff members and medical professionals to swiftly and effectively sanitize surfaces and equipment, and dramatically decrease the time between medical diagnostic equipment usage on patients.